



Medication Reminders

Operating Instructions  
e-pill® MedGlider HOME PLUS

SKU: 843742 UPC: 837066000633

Thank you for purchasing an  
e-pill Medication Reminder

We suggest you to read the following instructions carefully and keep them for future reference.

This easy-to-use rechargeable e-pill reminder can remind you to take your medications up to 4 times per day. It has a loud beeping alarm and visual indicator. This e-pill features a unique indicator that counts down to your next alarm time. The e-pill Medication HOME PLUS allows you to set individual alarm times for each day of the week.

**e-pill®** Medication Reminders

e-pill, LLC  
49 Walnut Street, Bldg. 4  
Wellesley, MA 02481, USA  
1-800-549-0095(+1 781 239 2941)  
Fax +1 (781) 235-3252

[www.epill.com](http://www.epill.com)

Version 170907

# Setting up Your e-pill® MedGlider HOME PLUS

Please activate the timer by removing the plastic bag from the rechargeable battery located in the battery compartment.

## Setting the Time and Day

Default screen as shown



### Day:


1. Press and hold down the **SET** button until the Day of the Week starts flashing.
2. Use ▲ and ▼ to adjust to current Day.
3. Press **SET** to confirm.

### Time:

1. The Hour starts to flash after Day is confirmed.
2. Use ▲ and ▼ to adjust to current Hour.
3. Ensure **AM** or **PM** is correct.
4. Press **SET** to confirm.
5. The Minute starts to flash.
6. Use ▲ and ▼ to adjust to current Minutes.
7. Press **SET** to confirm.

\*Nothing should be flashing after the time is set.\*

## Setting the Alarms

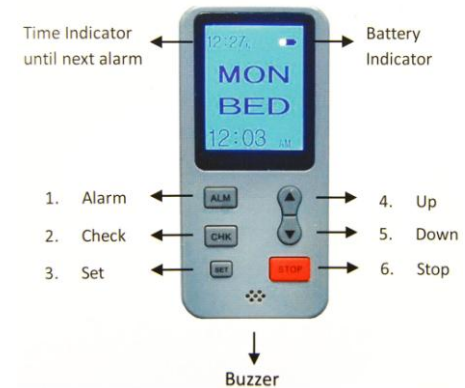

### Alarm:

1. Press and hold down the **ALM** button until the Day of the Week starts flashing.
2. Press ▲ and ▼ to adjust the Day of the Week.
3. Press **SET** to confirm.
4. Day Segment (MORN, NOON, EVEN, BED) will start to blink.
5. Press ▲ and ▼ to choose Day Segment.
6. Press **SET** Hour will start to flash.
7. Press ▲ and ▼ to adjust to the Hour.
8. Press **SET** Minutes will start to flash.
9. Press ▲ and ▼ to adjust the Minutes.
10. Press **SET** to save alarm.

\*Repeat Steps 1-10 for each Day Segment & Day of the Week. It allows you to set different alarms for every day.\*

# Daily Use

## Turning Off Alarms

There are two types of alarm notification:

1. Flashing screen
2. Alarm sound

## Snooze

This reminder has a SNZ (Snooze) feature. The first time you press the **STOP** button, it will stop the alarm and SNZ will show up on the top left corner of the screen. The alarm will come back on 5 minutes later.

\*If you do not press the **STOP** button to stop the delay alarm, the alarm will go off for 30 minutes. After 30 minutes, the screen will keep flashing until the **STOP** button has been pressed for the second time.

You can turn off the alarm directly by pressing the **STOP** button twice (No Snooze).

## Checking the Alarms

1. Press the **CHK** (Check) button.
2. **CHK** will appear on the top left corner of the screen.
3. Press **▲** and **▼** button to check each alarm.
4. Press **STOP** to exit.



## Removing Alarms

1. Press and hold down the **ALM** button until the Day of the Week starts flashing.
2. Choose the alarm that you would want to remove by pressing **▲** and **▼** button.
3. Press **SET** button twice and Hour starts to flash.
4. Simply press and hold down the **CHK** button to remove the alarm. “(--:--)” should appear.
5. Press **STOP** to return to the current Day and Time.



\*Any time pills are missed, you should follow the directions from your physician or pharmacist before taking any medication (pill) out of sequence.\*

## Low Battery Alert

When the battery level is lower than 25% the battery icon on the top right corner will start to flash to indicate a low battery level. Recharge the battery by using the included AC Adapter.

Do not use any other AC Adapter to recharge your e-pill MedGlider HOME PLUS.

We recommend that you charge your timer each day.

## Disclaimer

This e-pill Medication Reminder is only a supplement of any memory system that you are currently use to remind yourself to take medications. The user maintains full responsibility to administer medications at the appropriate times. e-pill, LLC assumes no responsibility for any incidental or consequential damages, including but without limitation, damages or injuries resulting from malfunction of the product. e-pill, LLC’s liability shall in no circumstances exceed the amounts paid by the user for the timer.