



Need help?
e-pill Medication Reminders
1-800-549-0095

Monday – Friday
8:30am – 5pm Eastern

You will need:

1. The new inhaler canister
2. PuffMinder Doser
3. A pen or paperclip

Setup 1:

1. Place PuffMinder Doser on top of the inhaler canister
2. Smooth down the unit's rubber collar around the inhaler canister

Setup 2:

1. Turn on PuffMinder Doser
 - a. With the pen or paperclip, press and release the "CLEAR" button, the unit will wake up and displays "0 0". This indicates 0 doses remaining and 0 doses taken today.
2. Set the clock:
 - a. Press and release the "SET" button
 - b. Press and release the "HISTORY" button
 - c. Press and hold (or tap) the "SET" button until you reach the hour that is closest to the current time
 - d. Wait a few moments and the screen will go back to "0 0"

Setup 3:

1. Set the number of inhalations:
 - a. Look at the label of the canister to identify the number of inhalations
 - b. Press and release the "SET" button and the number on the screen will flash
 - c. Press and release the "CLEAR" button, and the number will continue to flash
 - d. Press and hold (or tap) the "SET" button, until you have reached the number of inhalations in your inhaler canister
 - e. Wait a few moments for the screen to stop flashing

PuffMinder Doser is now set to count and keep track each puff of medication.

When replacing your inhaler canister:

Follow "Setup 3" to reset PuffMinder Doser at your next full inhaler cannister