




# “Average” Presbyopic Additions

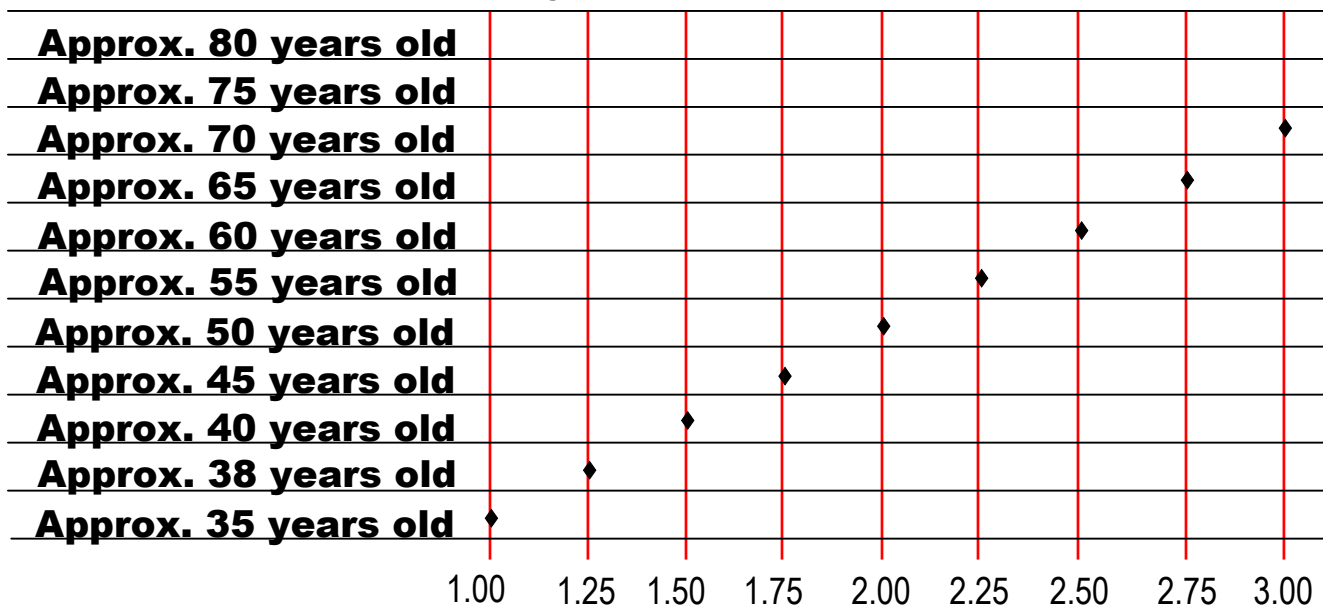

Determining your power is easy using either of the following methods:

1. Call 1 - 888 - MULTI - VIEW (1 - 888 - 685 - 8484) service line and speak with one of our Opticians.
2. Ask your Doctor which reading power is best for you, from the power selection chart.
3. Print this page, Without wearing glasses, hold this paper 14” away from your face. Start with the largest print size at the bottom of the page and work your way up. The power to the right of the line that becomes difficult to read, is the power you should order.
4. Custom Multi-Views are available if we fill your exact prescription or use a frame that is not in the Multi-View product line. For example, the use of your own frame. Please call toll free to place those custom orders with one of our Opticians. Ask our Opticians about Multi-Views with a sun tint for outside reading or an anti-reflective coating to reduce glare.

It's as easy as that!

**Power Selection Chart**  
**Start From the Bottom**

Power  
to Order

If this is the first line that is not clear, I need	➡	<b>+1.00</b>
If this is the first line that is not clear, I need	➡	<b>+1.25</b>
If this is the first line that is not clear, I need	➡	<b>+1.50</b>
If this is the first line that is not clear, I need	➡	<b>+1.75</b>
If this is the first line that is not clear, I need	➡	<b>+2.00</b>
If this is the first line that is not clear, I need	➡	<b>+2.25</b>
If this is the first line that is not clear, I need	➡	<b>+2.50</b>
If this is the first line that is not clear, I need	➡	<b>+2.75</b>
If this is the first line that is not clear, I need	➡	<b>+3.00</b>